


Kronner Vaginal Cuff Manipulator, (VCM)

For supporting the vagina during laparoscopic vaginal procedures

Hand held or Attached to the Kronner Sidekick uterine manipulator holder through the Vcare® adapter

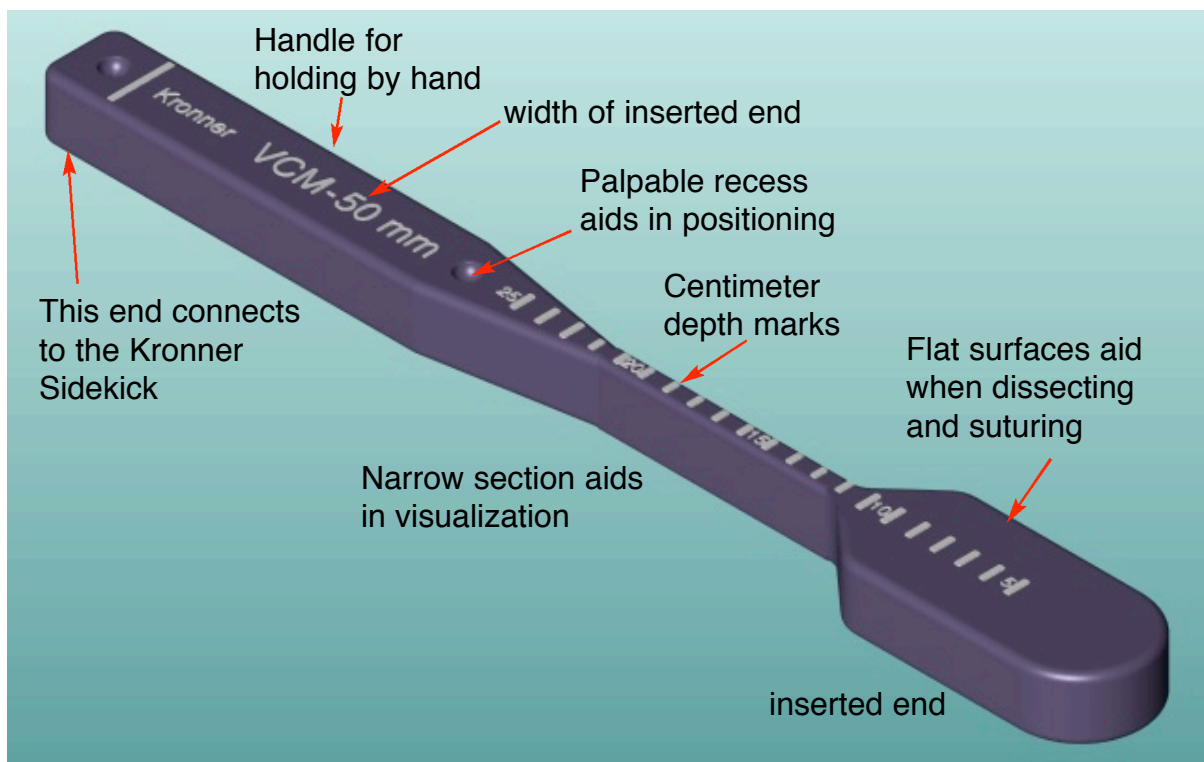

Sizes available:

- 40 mm width inserted end
- 45 mm width inserted end
- 50 mm width inserted end

Cleaning and sterilization instructions for the VCM

1. Use **enzymatic detergent**, which is protein solubilizing, safe for use with metal instruments, and prepared according to the manufacture's recommendations.
2. **Soak the VCM** for five minutes.
3. **Agitate, and scrub the submerged part** with a soft bristled brush. Maneuver movable parts to loosen trapped soil.
4. **Rinse the VCM** with warm tap water at 38-49° C.
5. **Agitate the VCM by hand** for at least one minute in a bath of warm water at 38-49° C.
6. **Repeat steps 4 and 5 two additional times.**
7. **Rinse the VCM** with clean tap water for at least one minute.
8. **Dry the exterior of the VCM** with a clean, lint free cloth.

Sterilizing the components

1. Place items in a Sterilizer tray
2. Sterilization wrap : Kingard KC400
3. Pre-vacuum temperature: 270 F, (132C)
4. Exposure time: 4.0 minutes
5. Drying time: 45 minutes

To order:

Vaginal cuff manipulators

- | | |
|--------|-------------|
| VCM-40 | 40mm width |
| VCM-45 | 45 mm width |
| VCM-50 | 50mm width |

Kronner Medical
1443 Upper Cleveland Rapids Road
Roseburg, Oregon 97471
Phone: (800) 706-3533 or (541) 672-2543
Fax: (541) 672-1074
E-mail: kronner@rosenet.net Website: www.kronner.com