



Using the Vaginal Cuff Manipulator, (VCM)

following the use of the Vcare® uterine manipulator

1. Loosen the Vcare joining screw.
2. Support the Vcare® manipulator, release the gas and remove the Sidekick from the Vcare manipulator.

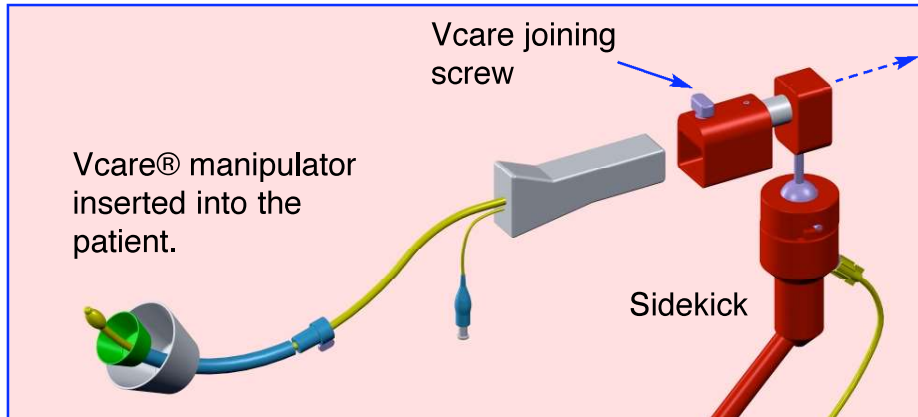


3. Re-apply the gas by removing pressure on the gas pedal.
4. Remove the manipulator from the patient.

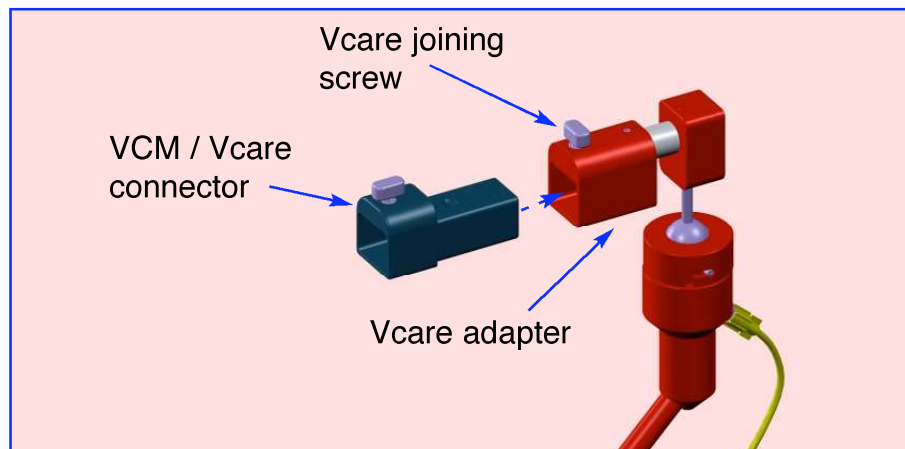


5. With the Vcare joining screw partially withdrawn, insert the VCM / Vcare connector into the VCare adapter as far as possible.


6. Tighten the Vcare joining screw.

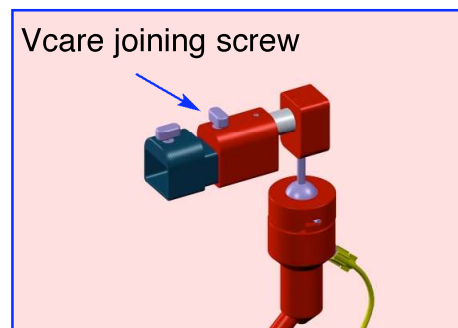

7. Insert the VCM into the patient.
8. Support the VCM.
9. Support Sidekick and release the gas by stepping on the foot pedal.
10. With the VCM connection screw partially withdrawn, join Sidekick to the VCM.

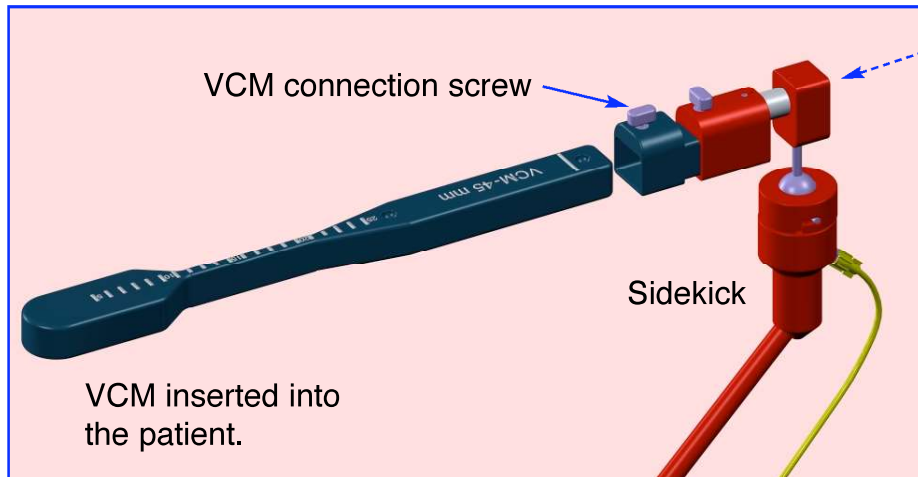

11. Re-apply the gas by removing pressure on the gas pedal.

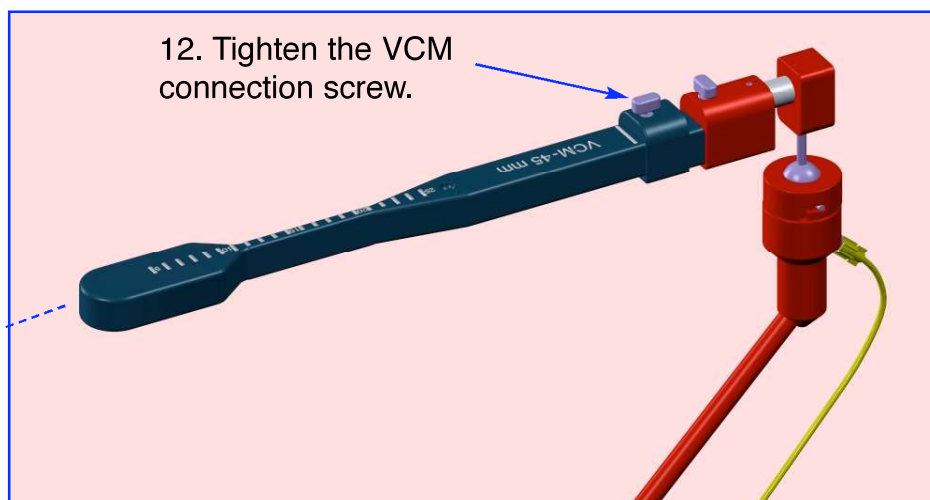

13. Grasp the VCM to rotate. It is not necessary to release the gas.
14. To reposition the VCM, grasp the VCM handle and release the gas.