


# The KRONNER SIDE-KICK

for holding the Arch Koh uterine manipulator

## Abbreviated Instructions for use

for detailed instructions see the INSTRUCTIONS FOR USE booklet or CD

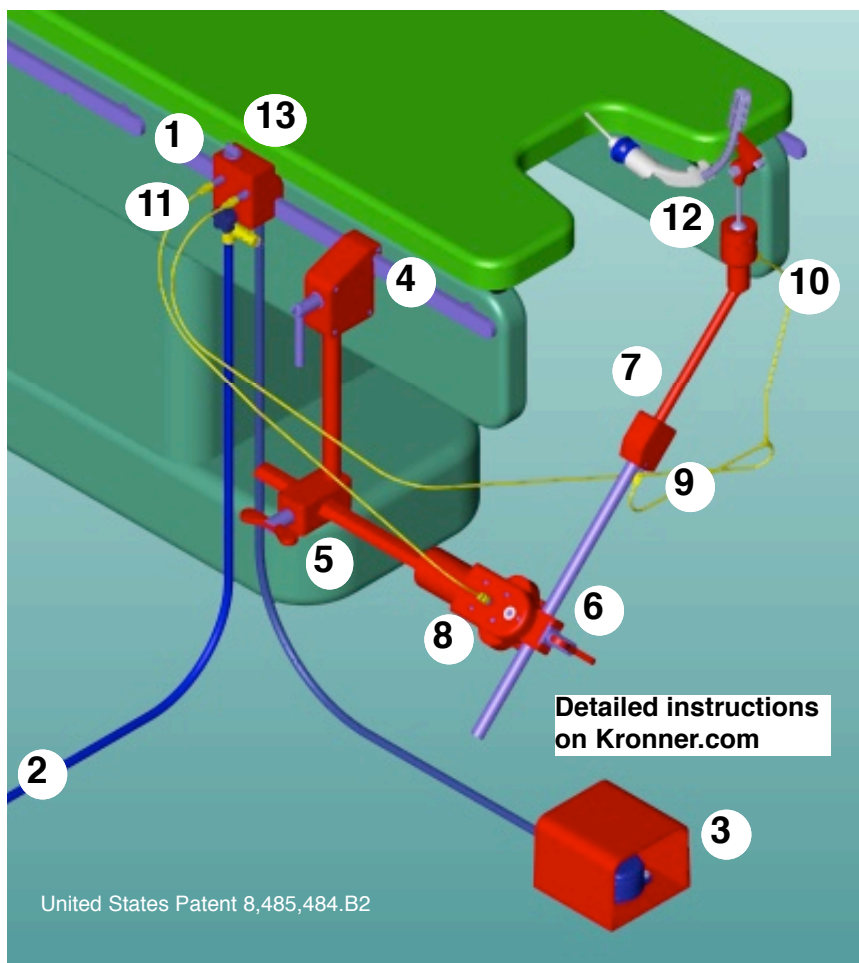


Kronner Medical, 1443 Upper Cleveland Rapids Road, Roseburg, OR 97471  
Tel: 541-672-2543

## Attaching the SideKick assembly to the operating table rail.

1. Attach the gas supply module to the operating table rail and turn off the module switch.
2. Connect the gas supply hose to the hospital nitrogen supply.
3. Position the foot pedal. **Warning: Do not attach a gas line to a uterine manipulator, extreme danger.**
4. Attach the rail grip to the rail.
5. Slide the pivot rod into the rail grip.
6. Insert the outer arm into the pivot.
7. Insert the inner rod into the outer arm.
8. Attach the single gas line to the pivot and the other end to the gas supply module #1.
9. Attach one branch of the branched line to the outer arm luer.
10. Attach the second branch of the branched line to the inner rod ball joint luer.
11. Attach the other end of the branched line to the gas supply module.
12. Join the adapter and attached manipulator to the inner rod, see Arch Koh instructions below.
13. Turn the gas module switch to "ON".
14. When repositioning the manipulator, step on the foot pedal #3.

## To Join the Arch Koh to the Adapter

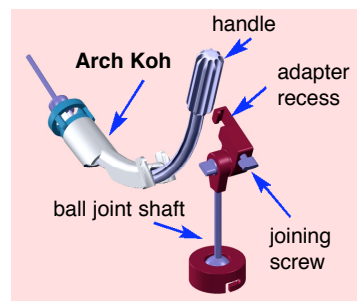

1. Retract the joining screw so the Arch Koh can be slid into the adapter recess.
2. With the ball joint shaft directed downward, slide the Arch Koh into the recess. This connection is made just below the Arch Koh handle.
3. Tighten the joining screw.
4. Insert the manipulator into the patient.

## Attaching the Adapter to the Inner Rod

1. Connect the adapter to the arm by sliding the two adapter notches over the two arm pins and rotating the adapter clockwise. Gas pressure must be off during this step.

