


Using the Kronner VCM With the Universal Adapter attached to the Sidekick

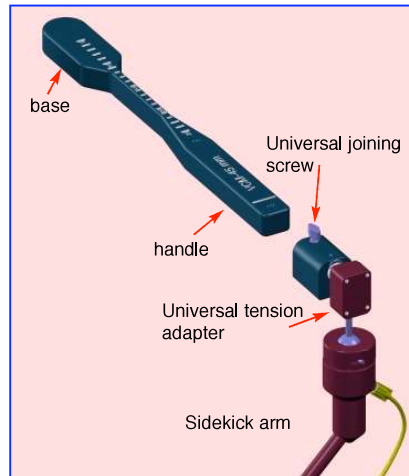

1. Select the VCM with the base insertion width of 40, 45, or 50 mm.
2. Loosen the joining screw on the universal tension adapter.
3. Support the the Sidekick arm and insert the VCM into the adapter.
4. Tighten the joining screw.

For VCare users, the VCM connects to the Kronner SideKick through the VCM / VCare Connector and VCare tension adapter.

The Kronner VCM / VCare radel connector fits between the VCare tension adapter and the Kronner VCM so the VCM can be held by Sidekick for vaginal surgery when the only adapter available is for holding the VCare® manipulator.


This procedure usually follows a laparoscopic hysterectomy aided by the Vcare® uterine manipulator.

The connector is non-conductive, preventing the VCM from any possible grounding and thus protecting the VCM from causing a patient injury from electrocautery.


The VCare tension adapter allows the VCM to be rotated 360 degrees around its long axis. Sidekick allows for unrestricted other positioning.

The Vaginal Cuff Retractor (VCR)

Hand or Sidekick held

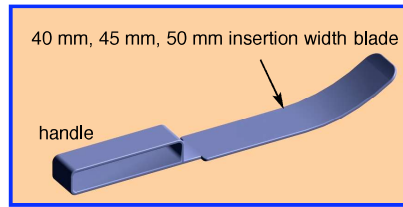

Using the Kronner Vaginal Cuff Retractor with the Universal Adapter attached to Sidekick

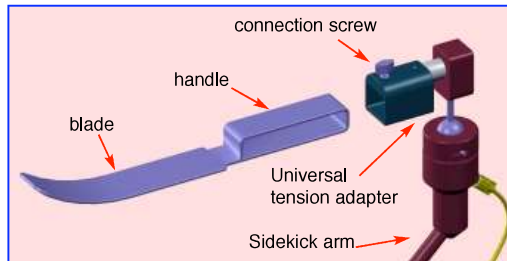

1. Select the 40 mm, 45 mm, or 50 mm vaginal cuff retractor
2. Insert the retractor into the patient
3. Loosen the connection screw on the black tension adapter.
4. Support the the Sidekick arm and insert the adapter over the retractor.
5. The retractor can be inserted after uterine manipulator and VCM removal.
6. Tighten the connection screw.

Rotating the Kronner Vaginal Cuff Retractor with the Universal Adapter

The attached retractor can be rotated 360 degrees without stepping on the foot pedal.

1. Grasp the handle or tension adapter and rotate.

